


** Please remember to protect flooring and the upholstery when turning a Plinth on the side **




Turn the Plinth on it's side, so that the pump upper-most.

Remove bolts "A" and "B", for this you will need two 19mm spanners.

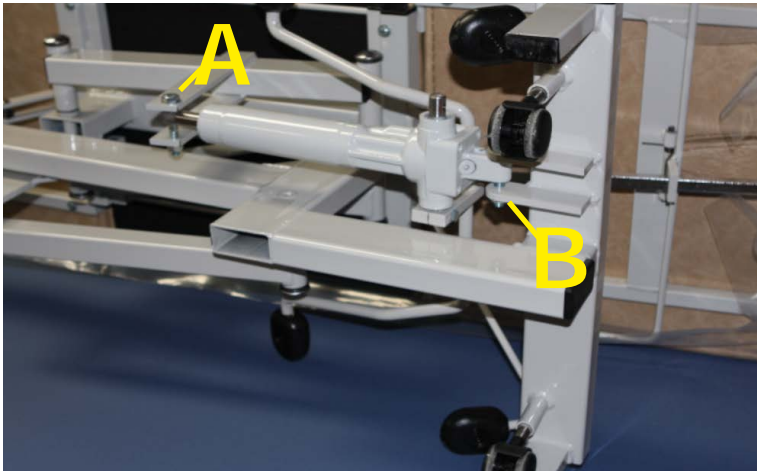

Remove old pump and pedal (which is attached).

Use two 13mm spanners to remove bolt "C" from existing pump pedal and re-fit to new pump, ensuring it is fitted to the same side as the drain screw (labelled "D" here).

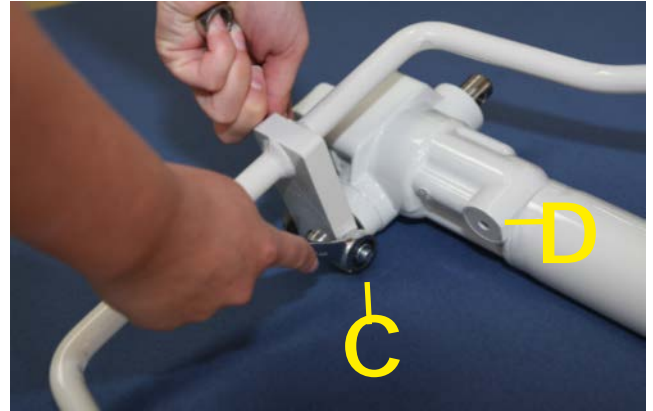

Fit new pump by securing rear bolt, "A", and pump until bolt "B" is in line with the holes on the front bracket. Tighten both bolts.